


















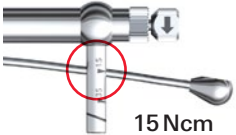


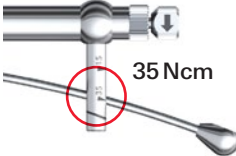


Prosthetic components

<p>Plastic Temporary Abutment Plastic Temporary Coping Healing Abutment Healing Cap</p>					<p>By hand</p>
<p>Prosthetic Screws for final restorations Titanium Temporary Coping</p>					 <p>15 Ncm</p>
<p>Ball Abutment 17° Multi-unit Abutment 30° Multi-unit Abutment</p>					
<p>Abutments for NobelActive® 3.0</p>  <p>Max 15 Ncm</p>					
<p>All other abutments for Nobel Biocare implant systems</p>					 <p>35 Ncm</p>

Note: Other major implant systems may require different torque values. Always consult the respective instructions for use.